SRN: DE-MF-000021120

Basis UDI-DI: ++EWPDCCKPLC

ARUM Peek

Whitepeaks Dental Solutions GmbH • Weikenrott 17 • 46499 Hamminkeln • Germany • +49 281 2064580

ARUM 3D Solutions LTD. • 26C Lyon Road, Walton-on-Thames • KT12 3PU • United Kingdom • +44 20 7388 1390

	Gebrauchsanweisung	2
	Instructions for use	5
	Manuel d'utilisation	8
	Istruzioni per l'uso	11
	Instrucciones de uso	14
	Instrukcje użytkowania	17
	Instruktioner för användning	20
	Instruktioner til brug	23
	Bruksanvisning	26
	Käyttöohjeet	29
	Gebruiksaanwijzing	32
	Οδηγίες χρήσης	35
	Upute za uporabu	38
	Instrucțiuni de utilizare	41
	Naudojimo instrukcijos	44
	Návod k použití	47
	Návod na použitie	50
	Használati utasítás	53

SRN: DE-MF-000021120

Basis UDI-DI: ++EWPDKPLC

ARUM Peek

Whitepeaks Dental Solutions GmbH • Weikenrott 17 • 46499 Hamminkeln • Germany • +49 281 2064580

ARUM 3D Solutions LTD. • 26C Lyon Road, Walton-on-Thames • KT12 3PU • United Kingdom • +44 20 7388 1390

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Produkt/ Produkttyp

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Instructions for use

Product/ Product type

PEEK blanks for the fabrication of individual dental restorations

Material type

Polyetheretherketone (PEEK) - Class IIa medical device

Type 3 denture base material according to DIN EN ISO 20795-1 (all color variants)

polymer-based crown and veneering material (type 2 class 1 according to DIN EN ISO 10477 (ARUM Peek Bio Classic, Light, Medium)

As these are thermoplastically produced CAD/CAM blanks, the thermoplastic molding process is not required by the user.

Users

Professional users who fabricate customized dental restorations/splints.

Target group

Patients of all ages and genders who require customized dental restorations.

Indication / Intended use

The product is a medical device intended for the manufacture of dental prostheses intended for long-term use, which are partially inserted into the human body by means of a clinical procedure, but in this case are inserted into the teeth or attached to the tooth structure with suitable fixing material and are therefore to be assigned to Class IIa.

Permanent, removable or partially removable (screw-retained) dentures can be fabricated. This means that crowns, bridges, denture bases, model casting dentures, transversal connectors, retaining elements, sublingual brackets, secondary parts, bars, attachments, hybrid abutments, superstructures for combined dentures, implant-supported full crowns in the posterior region and conditionally removable, screw-retained bridges can be realized with the material described.

Contraindication

Do not use in case of proven hypersensitivity to one or more ingredients.

Material properties/ Technical data

<u>Material composition:</u>	Bio Classic	Light	Medium	Rose
PEEK (polyether ether ketone)	≈ 100%	≈ 80%	≈ 80%	> 90%
Titanium dioxide (TiO ₂)		< 20%	< 20%	< 5%
Other inorganic pigments	< 0,1%	< 0,1%	< 1%	< 1%

<u>Technical data:</u>	Bio Classic	Light	Medium	Rose
Bending strength	~ 165 MPa	~ 180 MPa	~ 180 MPa	~ 175 MPa
Bending modulus	~ 4050 MPa	~ 4700 MPa	~ 4800 MPa	~ 4400 MPa
Water absorption	~ 4.66 µg/mm ³	~ 4.66 µg/mm ³	~ 4.66 µg/mm ³	~ 4.66 µg/mm ³
Solubility	~ 4.66 µg/mm ³	~ 4.66 µg/mm ³	~ 4.66 µg/mm ³	~ 4.66 µg/mm ³

SRN: DE-MF-000021120

Basis UDI-DI: ++EWPDCGPLC

ARUM Peek

Whitepeaks Dental Solutions GmbH • Weikenrott 17 • 46499 Hamminkeln • Germany • +49 281 2064580

ARUM 3D Solutions LTD. • 26C Lyon Road, Walton-on-Thames • KT12 3PU • United Kingdom • +44 20 7388 1390

Processing

The desired restoration is milled from the selected blank. ARUM Peek is to be processed with tungsten carbide burs. A coarse-toothed carbide bur must be used to separate the milled prosthetics from the milling blank. The retaining bars must be separated without applying pressure. The restorations can be polished with suitable silicone polishers, goat hair brushes and suitable polishing paste (diamond-coated).

Minimum wall thickness: 0.6 mm

Connector cross-sections in the anterior tooth area: 10 mm²Connector cross-sections in the side tooth area: 16 mm²

No more than two pontics may be positioned in the posterior region.

Fastening

After the try-in, a composite primer suitable for PEEK is applied to the 110 µm aluminum oxide blasted inner surface of the crown. The primer creates a secure bond between ARUM Peek and the luting composite. The instructions for use of the composite manufacturer must be observed.

Veneering

All commercially available veneering materials that are suitable for PEEK - The respective instructions for use of the manufacturer of the bonder, veneering or luting material used must be observed.

Bonding of plastic teeth

Plastic teeth can be attached to denture bases, model casting dentures or ARUM Peek transversal bars using a suitable luting material. The respective instructions for use of the manufacturer of the bonder, veneering or luting material used must be observed.

Safety instructions

Warning: Always wear respiratory protection (filter class FFP2), tight-fitting safety goggles, protective gloves and protective clothing and switch on extraction equipment. The dust generated during processing of this product may cause irritation to the skin/eyes/respiratory tract. Avoid inhalation/contact with skin, mouth and eyes. Do not eat or drink while working. Keep away from food and drink. Wash hands after handling. Remove contaminated clothing and protective equipment before entering eating areas. Keep away from sources of ignition. Do not smoke.

Caution! Hazardous respirable dust may be generated during use. Do not inhale dust. Safety data sheet available on request.

Storage

Store in a dry place. Protect from moisture. Store in the original packaging.

Waste disposal

Dispose of the product and packaging in accordance with local/ regional/ national/ international regulations.

Note

Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

The summary of safety and clinical performance according to (EU) MDR 2017 / 745 Article 32 (SSCP) is available in the European database for medical devices (Eudamed) under the corresponding base UDI-DI as soon as the corresponding database module exists.

<https://ec.europa.eu/tools/eudamed>

SRN: DE-MF-000021120

Basis UDI-DI: ++EWPDCPLC

ARUM Peek

Whitepeaks Dental Solutions GmbH • Weikenrott 17 • 46499 Hamminkeln • Germany • +49 281 2064580

ARUM 3D Solutions LTD. • 26C Lyon Road, Walton-on-Thames • KT12 3PU • United Kingdom • +44 20 7388 1390

Explanation of the markings on the packaging

Confirmation: The product complies with the applicable European directives

Catalog number/ Item number

Batch designation

Clear product identification

Number of products in the packaging

Observe the safety instructions in the operating instructions and instructions for use

www.wp-ifu.de/arum/peek

Path to the electronic instructions for use

Instructions for use in paper form available within 7 days

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