SRN: DE-MF-000021120
 Basis UDI-DI: ++EWPDCCLKC
 M+W SELECT LIQUID

Whitepeaks Dental Solutions GmbH • Weikenrott 17 • 46499 Hamminkeln • Germany • +49 281 2064580

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Instructions for use

Product/ Product type

Colouring solution for zirconium dioxide ceramics

Material type

Colouring liquid in various shades - Class IIa medical device

Users

Professional users who fabricate and colour individual dental restorations.

Target group

Patients of all ages and genders who require customised dental restorations.

Indication / Intended use

The product is exclusively suitable for colouring the corresponding Whitepeaks zirconium dioxide ceramic in dental technology.

The product is a medical device intended for the colouring of dental prostheses intended for long-term use, which are partially introduced into the human body through a clinical procedure, but are inserted into the teeth or attached to the tooth structure with suitable fixing material and are therefore classified as class IIa.

Product	Intended use
M+W SELECT LIQUID	Colour liquid (dip dye) for colouring white/opaque zirconium dioxide ceramics before sintering
M+W SELECT LIQUID HT	Colour liquid (dip dye or brush dye) for colouring translucent zirconium dioxide ceramics before sintering
M+W SELECT LIQUID UHT	Colour liquid (brush colour) for colouring translucent zirconium dioxide ceramics before sintering

The type of coloured restoration depends on the respective Whitepeaks zirconium dioxide ceramic of the Copran product group. Please refer to the instructions for use for the Copran product group.

Contraindication

None

Material properties/ Technical data

The product is liquid, odourless, water-based, acid-free, free from oily substances, easy to use, easy to store, easy to dispose of and has a long shelf life.

Description

The product is exclusively suitable for colouring zirconium dioxide ceramics in dental technology. This colour matching has the advantage that the dental prosthesis can be individually adapted to the patient. When used as directed, the product is completely biocompatible and non-toxic. The product is water-based and contains no acids, so it does not attack metal instruments or work surfaces. The heating elements of the sintering oven are also not affected. The product does not contain any oily substances, can utilise the capillary effect of the zirconium dioxide and penetrates quickly and deeply into the material. No light-coloured spots or stains are created when separating sinter drops and connecting bars. If a colour is too intense, the colour liquid can simply be diluted with distilled water.

Processing

The restoration should be finished and dust-free. The restoration should be processed dry.

Preparation

In order to achieve a perfect surface and maximum transparency, we recommend cleaning the restoration in the restoration in the White-Sonic ultrasonic cleaning device and distilled water. Please only use our recommended ultrasonic cleaning device. Devices from other companies could be too 'strong' and damage the restoration. Swirl the restoration with plastic tweezers in the distilled water of the ultrasonic unit for 5 to 10 seconds until no more 'dust

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clouds' are detached from the restoration. Remove excess water from the restoration by blowing it off with oil-free air or by dabbing it dry with cellulose or cotton swabs.

Dry the restoration under an infrared lamp or using an oven. The restoration must be completely dry before sintering. Make sure that the water never boils during drying, as this may cause cracks in the zirconium dioxide. Take the required colouring liquid, make sure that the container is properly closed and shake it for a few seconds to ensure that there is no sediment.

Diving technology

Now immerse the restoration completely in the colouring liquid of your choice for approx. 45 seconds. Remove the restoration and remove the remaining liquid with a moistened brush, by blowing off with oil-free air or by drying with cellulose or a cotton swab. Close the paint container tightly and put it back. If a restoration is too large for the original container, pour the colouring liquid into a clean container and colour your restoration there. Then clean the container thoroughly with water and detergent.

Painting technique

Start the staining process with a brush. We recommend applying the colour in the following order: tooth neck, cusp tips/incisal edges, incisal areas, occlusal colour and finally dentin colour

Use the Blocker Liquid carefully and sparingly. It prevents other liquids from penetrating the restoration, so it must be applied before all other liquids. The colour of the zirconium dioxide remains unchanged.

After applying the effect shades, the rest of the restoration can be coloured with the corresponding dentin shade. You can also apply the dentin colour from the inside of the crown to achieve the illusion of a dentin core. With a little skill and training, you will quickly be able to achieve fantastic, vibrant results.

Sintering

Dry your restoration completely according to the above-mentioned procedure. Incomplete drying may result in colour changes and opacity of the zirconium dioxide. Then sinter your restoration according to the instructions for use of the zirconium dioxide material used.

After final sintering, the restoration is adjusted to the working model using the wet grinding technique with diamond-coated grinding instruments, if necessary. Sintered diamonds, corundum stones or tungsten carbide burs must not be used.

Safety instructions

Always wear respiratory protection (filter class FFP2), tight-fitting safety goggles, protective gloves and protective clothing and switch on the extraction system. Ensure good ventilation. Do not inhale vapour/aerosol. Avoid contact with skin, mouth, eyes and clothing. Do not eat or drink while working. Keep away from food and drink. Wash hands after handling. Remove contaminated clothing and protective equipment before entering eating areas. Protect from heat and sunlight. Keep away from sources of ignition. Do not smoke.

Warning: May contain Cobalt (II) Chloride Hexahydrate. May cause cancer if inhaled. Suspected of causing genetic defects. May impair fertility. Be sure to observe the above safety precautions.

May contain nickel (II) chloride hexahydrate. May cause cancer by inhalation. Suspected of causing genetic defects. May impair fertility or cause harm to the unborn child.

Shelf life/storage

3 years at room temperature and in an airtight bottle.

Store in the original packaging. Store in a dry place.

Waste disposal

Dispose of the product and packaging in accordance with local/ regional/ national/ international regulations. Do not dispose of with household waste. Do not allow to enter waterways, groundwater or sewage system.

Hint

All serious incidents related to the device must be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

The summary of safety and clinical performance according to (EU) MDR 2017 / 745 Article 32 (SSCP) is available in the European database for medical devices (Eudamed) under the associated base UDI-DI as soon as the corresponding database module exists.

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<https://ec.europa.eu/tools/eudamed>

Explanation of the labelling on the packaging

Confirmation: The product complies with the applicable European directives

Catalogue number/ item number

Batch designation

Unique product identification

Observe the safety instructions in the operating instructions and instructions for use

www.wp-ifu.de/mw/liquid

Path to the electronic instructions for use

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M+W Müller & Weygandt GmbH • Reichardsweide 40 • 63654 Büdingen • Germany • +49 6042 8800-88